

**SPECIFICATIONS**  
**LC-108**

SPECIFICATIONS	COLLECTION	ELEGANCE
SIZE		Medium -Large
MODEL NUMBER		LC108
POSITION		Infinite
HAND CONTROL		Easy Touch Infinite-Position with USB Port
BACK TYPE		Sewn Pillow Back
SEAT TYPE		Sinous spring seat
STANDARD FEATURES		Footrest extension
ADDITIONAL FEATURES		Revolvable back cushion and chaise pad
USER HEIGHT RANGE		5'4" -6'
SEAT TO FLOOR		19"
SEAT WIDTH		19.5"
SEAT DEPTH		20"
SEAT DEPTH WITH FOOTREST ELEVATED		38.5"
TOP OF BACK TO SEAT		30"
CHAIR WIDTH		30.5"
CHAIR DEPTH		36.5"
CHAIR HEIGHT		48"
DISTANCE FROM WALL		26.5"
CHAIR WEIGHT		136 lb
WEIGHT CAPACITY		375 lb
WARRANTY		7-yr. pro-rated

FABRIC SELECTION GUIDE	STANDARD	Cloud 9	Black Cherry	Pacific	Stone	Walnut
	OPTIONAL	Lexis Sta-Kleen <sup>®</sup>				

**BENEFITS**

- Independant operation of the backrest and legrests allowing for fully customizable comfort
- Wooden frame composed of engineered furniture grade laminate for added strength and durability
- Quiet & smooth lift system
- Heavy-duty lift actuator & scissor mechanism
- Standard head & arm covers
- Fire retardant (California specifications; bulletin 117-2013)
- Standard, large dual pockets
- Removable back
- Integrated battery backup in case of power failure<sup>1</sup>
- Transformer features self-diagnostic electronics with 9' cord
- Lifetime warranty on steel lift frame

**OPTIONS\***

- Residential cleaning kit
  - Head pillow
  - Lumbar pillow
  - Non-skid leg levelers
- Additional lead time may apply for some options.*

<sup>\*</sup>Additional charges apply for all options.  
<sup>1</sup> Batteries not included.



Engineered for comfort and durability, the LC-108 Infinite- Position Power Lift Recliner from Pride<sup>®</sup> reclines to a full reclined position that elevates the legs above the head for maximum comfort and relaxation. The stylish LC-108 is available in eye-catching fabric selections sure to enhance your home's décor.

**LC-108**

*Designed for ultimate comfort and adjustability to meet individual needs*

The LC-108 is equipped with adjustable and removable components to allow for a custom fit and easier care and maintenance of the product.

### Adjustable, removable back cushion

- **The individual pillows of the back have zippers** to access the fiber fill stuffing. The stuffing can be added or taken away to customize the fit and comfort of the back cushions. This is especially helpful with accommodating a user that has a kyphosis. For users with longer legs, stuffing can be removed from the back cushions to increase seat depth, offering better overall comfort and support for the legs.
- **The individually tethered pillows** of the back allow them to move with the client when going into or coming out of recline to reduce shearing and shifting.
- **The standard wing back design** makes it possible to insert additional support along the side to address issues with hemiplegia or stroke. The wings also work as a safeguard to keep users from falling beyond the sides of the backrest.



WING BACK DESIGN



REMOVABLE / ADJUSTABLE  
BACK CUSHION



ZIPPERS TO ACCESS  
FIBER FILL STUFFING



INDIVIDUALLY TETHERED  
PILLOWS

### Adjustable, removable chaise pad

- **The chaise pad is fully removable** and the foam fill can be removed. This will allow for hand washing and hang drying of the seat cushion should it become soiled. The foam fill must be removed during washing. Moisture can cause damage to the foam.
- **Access to the seat cushion** also gives the ability to insert seating options to address pressure or comfort concerns.
- **The full chaise pad** provides a continuous support surface for the entire length of the seating surface, reducing pressure points and allowing for maximum comfort and relaxation.
- **Seating wedges or components** can be put between the seat box and chaise pad to correct postural seating concerns.



FULLY RECLINED WITH  
BACK AND CHAISE CUSHIONS



REMOVABLE CHAISE PAD



BACK AND CHAISE  
CUSHIONS REMOVED

### Additional Features


- **Both the seat and back cushions are replaceable** as parts should it be necessary due to wear and tear, increasing the life of the chair. (Note: Over time fabric fades and with different dye lots of fabric, replacement parts can sometimes differ slightly in color).
- **Push-button hand control standard** that allows for independent positioning of the leg rest and back angle. This control is also equipped with an all-up button that, when depressed, will lower the legs, raise the back and then lift the chair to the transfer position all in one continuous motion. The hand control has a quick disconnect feature in the side pocket to disable the functions of the control. There is also a built in USB charging port for portable devices.
- **Low-shear recline back standard provides** minimal shifting when going into recline and less forward force on the client when coming out of recline. The tethered pillows shift reducing the shear affect.
- **Fully landscaped arms** provide a continuous, even arm support surface with no ridges or transitions in height. This offers better comfort and arm support.
- **The standard footrest extension** gives more support to the feet, reducing pressure on the back of the calves and increasing relaxation to the feet. This often helps address cold feet issues.
- **The full-recline position with the feet fully elevated** allows for maximum comfort and is ideal for napping. The feet can be elevated above the upper body in this fully open position.
- **Standard arm and headrest covers** are supplied with the chair. This allows for easier care and maintenance of the chair and increases fabric protection.
- **Optional headrest and lumbar pillow** can be added to the chair to further enhance comfort and positioning.



FULL-RECLINE POSITION



SHOWN WITH STANDARD  
HEAD AND ARM COVERS

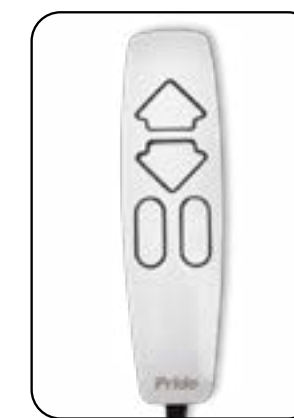

EASY TOUCH  
INFINITE-POSITION  
HAND CONTROL  
WITH USB PORT