

TYPE D CUP HOLDER INSTALLATION INSTRUCTIONS

Tools needed:

- Phillips screwdriver

If you require any assistance with these instructions, please call **Pride Technical Service** at **1-877-800-1248**.
Canada providers, phone **1-888-570-1113**.

Turn off the power to the controller and ensure the power base is in drive mode before beginning this procedure. Refer to the freewheel operation label on the power base.




1

WARNING! The Type D Cup Holder should be installed by an authorized service technician only.




2

WARNING! Ensure that the Type D Cup Holder DOES NOT mount to the armrest containing the controller!

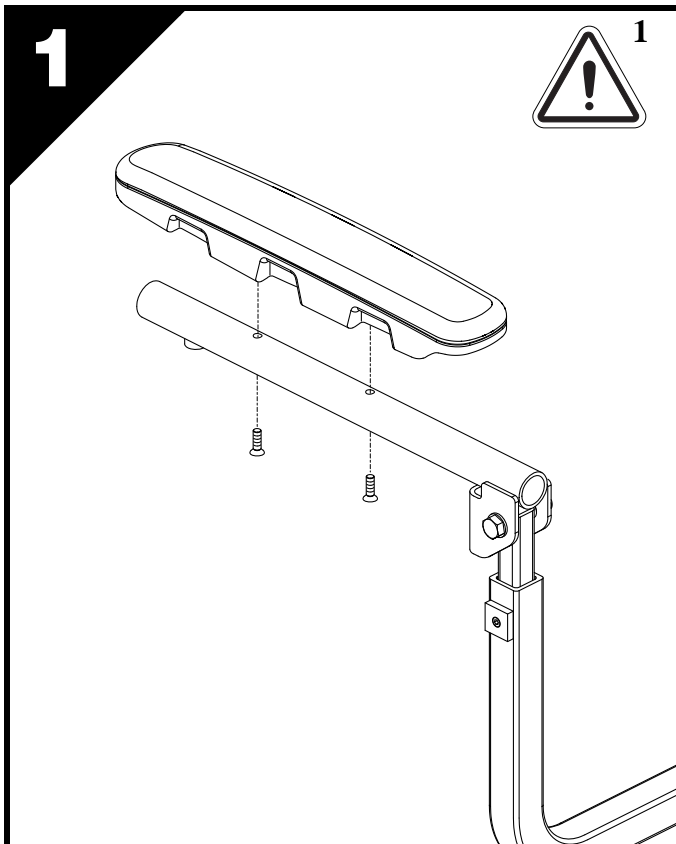

Remove the armrest cover and the accompanying securement screws from the armrest that **DOES NOT** contain the controller. Retain the armrest cover and discard the securement screws.

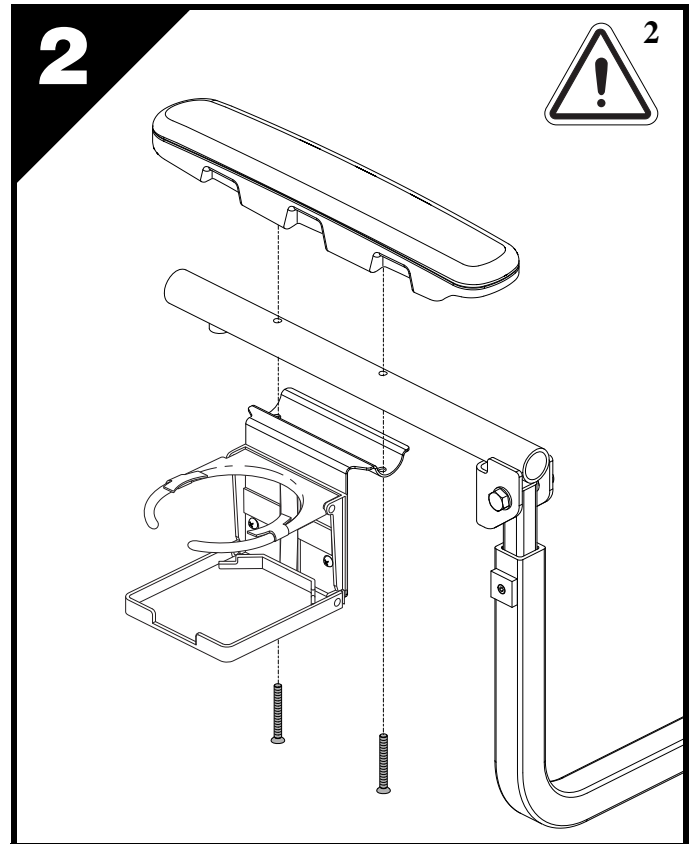

Install the Type D cup holder onto the armrest using the supplied securement screws and the retained armrest cover.